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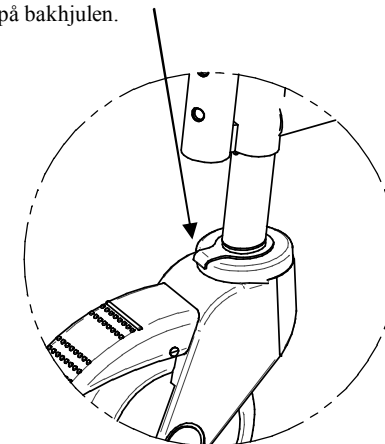
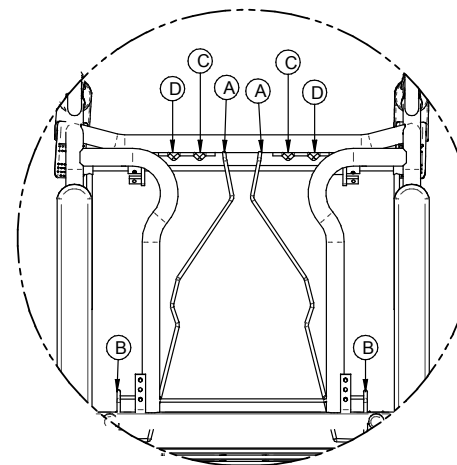
Directions for use

COMBI CHAIR. ATTENDANT OPERATED

Please read the instructions before you start using this product
Please keep the user instructions for future reference

These instructions apply to the Combi Chairs with and 4 braked wheels, to Combi Chairs with and without a bucket, and to Combi Chairs with and without the Transit Seat. All Combi Chairs have bedpan brackets. **The complete Combi Chair is made from rustproof materials** – including the 4 braked castors.

Purpose

The Combi Chair is a very versatile chair. It can be used as a commode, a shower chair, and a transport chair for indoor transportation. The Combi Chair has fold up armrests, a useful feature for both caregiver and user in situations of patient assistance and transfer. The Combi Chair is particularly suitable for persons who need assistance with toileting and with personal hygiene. The seat, backrest, and armrests are made of black polyurethane, a padded material with good anti-skid properties.

Assembly – see drawing

Remove the snaplocks from the frame and mount the backrest into the holes in the frame. Replace the snaplocks making sure that they are correctly inserted into the tubing. When the Combi Chair is used as a commode chair over a toilet, the backrest should be detached.

Adjustment – see drawing

By re-positioning the snaplocks on the legs, the Combi Chair is height adjustable in 6 steps. Remove the snaplocks one by one and select one of the 6 possible positions.

Make sure that the four legs have the same height and that all of the snaplocks are inserted correctly into the tubing again.

Make sure the red dot on the castors is facing outwards and that all 4 castors have the same height. The bed pan / bucket frame can be used for key hole shape bucket and round shape bucket. The two end pieces (B) rest on the rear of the seat frame. For key hole shape bucket, insert the two front rods (A) into the inner holes (C). For all other bucket shapes use the outer holes (D). If no bucket or bed pan is being used, then the frame can be detached.

Maintenance

To be wiped clean after use. Clean the Combi Chair with domestic cleaning materials without abrasive or corrosive properties. Solvents and chlorine should not be applied. Seat and backrest can be detached for cleaning purposes. The polyurethane material can tolerate temperatures of up to 70°C. Make sure to reattach all removed parts correctly. If alcohol is used to clean the black polyurethane, a certain discolouration can occur. The bucket / bedpan holder can be detached for cleaning purposes. Please note that the rear section of the bucket / bedpan holder must be lifted upwards **before** removing the two rods from the holes.

Caution

- Max. load. Combi Standard: 150 kg / 330 Lbs, Combi Wide: 200 kg. / 440 lbs
- Check the condition of the snaplocks frequently. Regularly check that the snaplocks are inserted correctly into the tubing.
- To detach the bed pan holder lift up the metal rods at the front and back.
- It is advisable to lock the wheels when the Combi Chair is not being used for transportation.
- If the castors are detached, it is important to mount them again correctly. Make sure the metal plates are facing outwards, and all 4 the castors have the same height.

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