TANGO

manual

Art nr: 74165A 03-09-18

TANGO

Svenska

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TANGO WALKING FRAME

TANGO is a foldable four-wheel walking frame for both indoor and outdoor use with swivelling wheels at the front.



TANGO has been tested and approved for a user weight of max 125 kg.

FOLDING

Folding: Apply the parking brake. Fold together the walker by pulling the red handle upwards. Avoid using the walker as support during the folding process.

Unfolding: unfold by pressing down the seat's frame.

Note! The seat must be properly in place when using the walking frame. The seat is properly in place when you hear a click.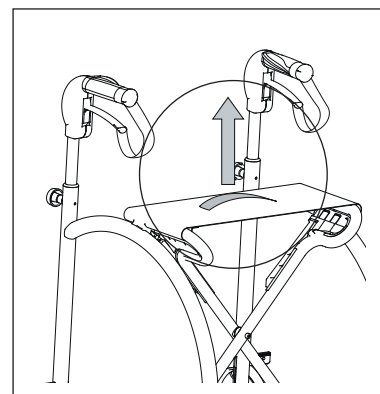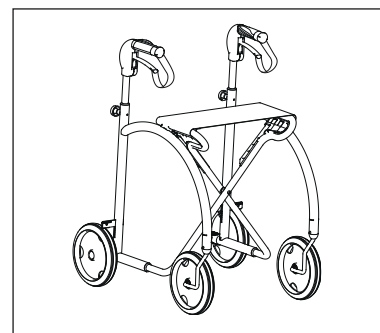


HEIGHT ADJUSTMENT

The handles are adjustable in height from 76 cm to 98 cm. When the height is tested the height of the upper part of the handles should match the height of your wrists (see picture).


Loosen the knob (1). Turn the handles inwards (2). Adjust the handles (3) to required height and tighten the knob.

If the walking frame feels unstable after use check that all knobs are properly tightened.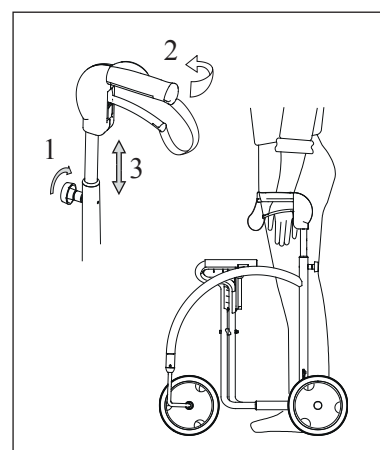

BRAKES: HOW TO USE


The brakes can be used in two different directions. If the brakes are pulled upwards TANGO brakes for as long as you hold the brakes. If the brakes are pressed downwards, the parking brake is activated. To release the parking brake just pull the brake upwards again and the brake is released.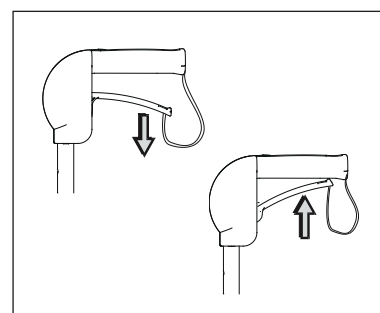

ADJUSTING THE BRAKES


When adjusting the brakes, loosen the screw and move the adjusting splint.

The distance between the adjusting splint and the tyre should be approx. 7 mm.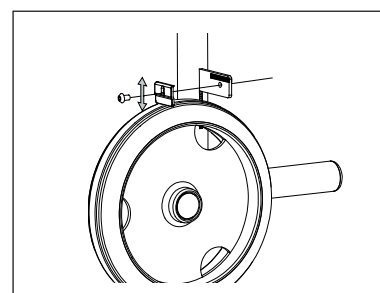

The brakes are adjusted at the factory. PLEASE NOTE...

- When you use TANGO it must be fully unfolded.
 - When the basket is fully laden you must be careful as the driving properties can be changed. This is particularly important when passing over thresholds or the edge of the pavement.
 - Max load in the basket is 10 kg.
 - When you sit on the seat the brake must be in the parking brake position.
 - The walking frame must not be used to transport people.
 - When using stairs the walking frame should be folded.
 - When sitting on the walking frame and moving it backwards or forwards using your legs please take extra care. Avoid sitting on the walking frame when going over the edge of rugs and thresholds.
 - When using on a sloping surface please take extra care.
 - The walker can easily negotiate pavements by using the tilter or by lifting the front with help of the cross tube.
- NB. Do not lift the walker by the seat.**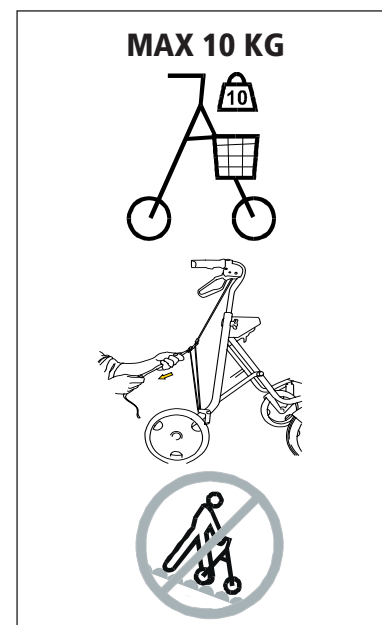

TANGO ACCESSORIES


Tray

Walking stick holder: 2 parts, available for right and left hand side.

Basket

Oxygen holder

Back support When the backrest is assembled and the height of the pushhandles set at 76 or 79 cm, the walker can be folded only to a width of 28cm.

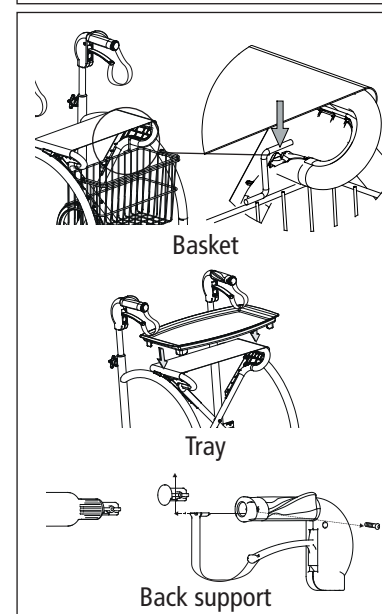LENGTH OF LIFE

The walking frame has been tested and fulfils the requirements in accordance with prEN ISO/DIS 11199-2. The durability test according to this method corresponds to a lifespan of 5 years at normal use.

SERVICE AND MAINTENANCE

- Clean the walking frame with car shampoo or washing up liquid.
- Max washing temperature 80°C.
- Do not use any solvents when cleaning.
- TANGO walking frame can be washed in a machine.
- If your walking frame feels unstable or does not work properly – contact the person who adjusts the equipment (retailer). A faulty walking frame must not be used.
- Do not tamper with the walking frame yourself.
- ETAC is not liable for damage to walking frames which has occurred through adjustments, temporary repairs, etc., carried out without its knowledge.

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